

THRIVE

Participant Guide



DOWNLOAD AND LOGIN

1 Download your designated app


Open your phone's camera and point it at the QR code.

When the website link appears, tap on it to open and download the app you were assigned to.




2 Log into the app

Type the username and password provided to you by your supervisor.

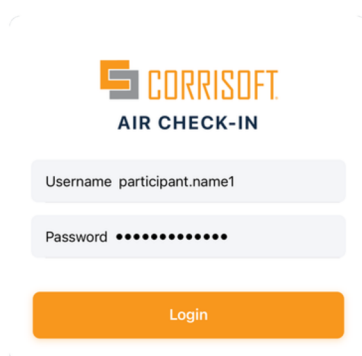

3 Accept all permissions

Make sure you allow all the requested permissions. If you miss granting any required permission, a red card will show up on your app's homepage.

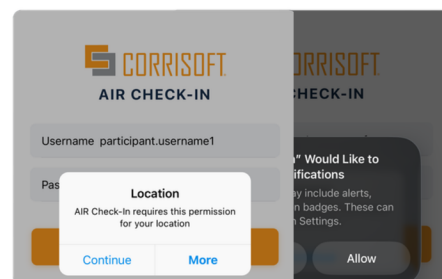

PREPARE BREATHALYZER

1 Insert Mouthpiece

Insert a mouthpiece into the mouthpiece slot on i70 pro breathalyzer device.



2 Turn the Device On

Press the square button at the front of the device to turn it on.



3 Device is "Ready to Pair"


Once the device is powered on, it should display the message "Ready to Pair."



TAKE A TEST

1 Open the test

Make sure the Breathalyzer is turned on and set to "Ready to Pair." Then, click the orange (or red) "BAC Test" pending check-in on the homepage.

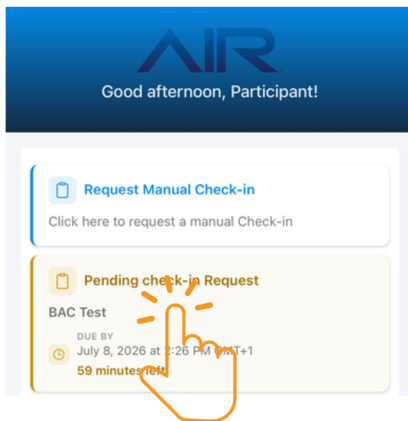


2 Device states "App"

After opening the test, the breathalyzer should say "App".




3 Start Test

Position your face on the screen and click the blue "Start" button.

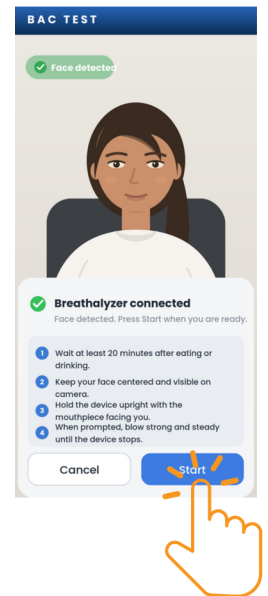


4 Wait for the countdown

Please wait until the breathalyzer countdown finishes before you begin blowing into the device.

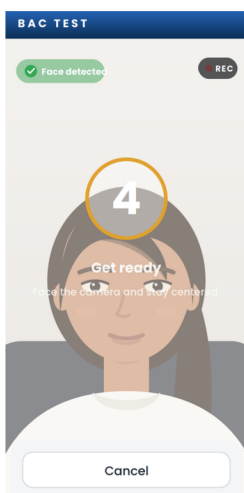

5 Start Blowing

When the app says "Blow Now," blow into the mouthpiece until you hear a "click" or the app starts "Analyzing the sample." Make sure your face stays in the frame during the test.

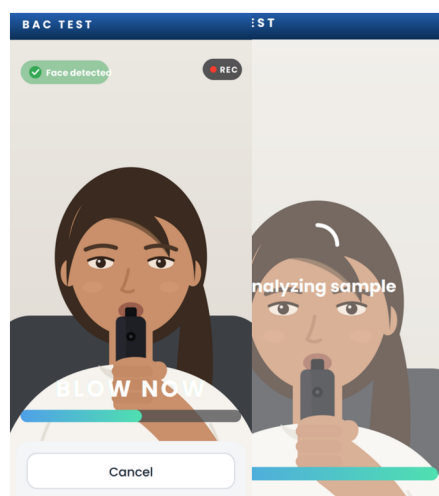

6 Wait for the Results

After you finish the test, you'll see your results on the screen. Click "Close" to go back to the Homepage.

